


Dengue Duo Cassette Test: fast and convenient


Procedure

Step 1

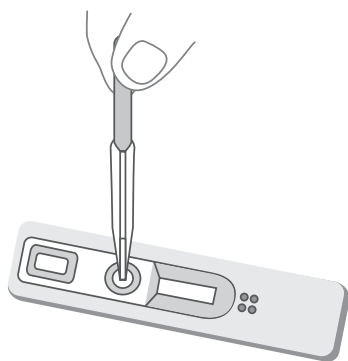


Add whole blood, serum or plasma to circular well

Step 2

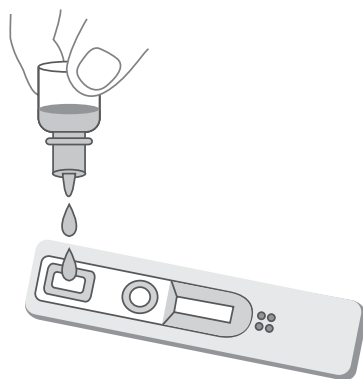


Add 2 drops of buffer to square well

Step 3

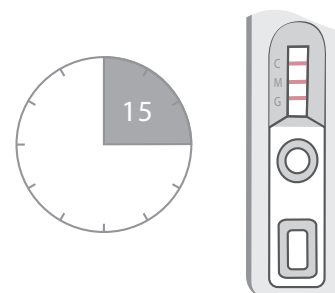


Read result in 15 minutes


Interpretation

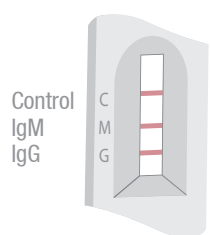


IgM and IgG positive
Secondary dengue

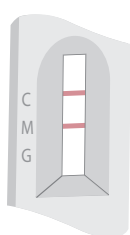

IgM positive
Primary dengue

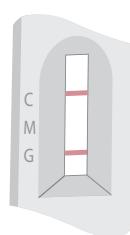

IgG positive
Secondary dengue

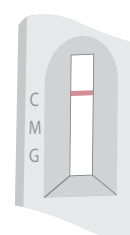

Negative

Ordering information

| Catalogue number | Description | No. of tests |
|------------------|---------------------|--------------|
| R-DEN03D | Dengue Duo Cassette | 25 |